

Front Side

Instructions for Use

SYMJEPI (slm-JEP-ee)
(epinephrine injection, USP)
pre-filled syringe

For Allergic Emergencies
(anaphylaxis)
**Read this instructions for
Use carefully before you use
this product.**


Before you need to use your
SYMJEPI syringe, make sure your
healthcare provider shows you the
right way to use it. Parents,
caregivers, and others who may be
in a position to administer SYMJEPI
should also understand how to use
it. If you have questions, ask
your healthcare provider.

Get Ready to Use SYMJEPI

SYMJEPI works
like a standard
pre-filled syringe.




SYMJEPI is injected downwards,
into the **thigh**, while sitting (as
shown), through clothing if
necessary.

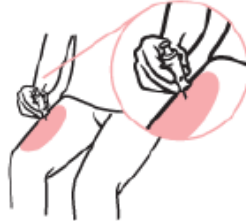

1

When ready to
inject, pull off cap
to expose needle.

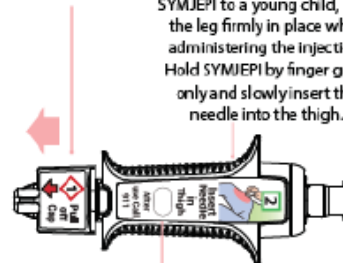

Solution window

2

Ready to Use SYMJEPI

**Do not put finger on top of
the device.**
If you are administering
SYMJEPI to a young child, hold
the leg firmly in place while
administering the injection.
Hold SYMJEPI by finger grips
only and slowly insert the
needle into the thigh.

3

After needle is in
thigh; Push the
plunger all the
way down until it
clicks and hold for
2 seconds.



Remove the
syringe and
massage the
injection area for
10 seconds.


Get Help

Get emergency
medical help now.



Tell healthcare
provider that you
received an
injection of
epinephrine.

Use second
syringe if
necessary

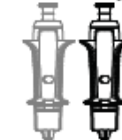

You may need
a second
SYMJEPI
syringe if
symptoms
continue or
recur.

Back Side

SYMJEPI is a single use pre-filled syringe that delivers a fixed dose of epinephrine. The pre-filled syringe cannot be reused. It is normal for most of the medicine to remain in the syringe after the dose has been injected.

The correct dose has been injected if the plunger has been pushed all the way down and the solution window is at least partially blocked.

If you will be administering SYMJEPI to a young child, ask your healthcare provider to show you how to properly hold the leg in place while administering a dose.

After use and Disposal - Hold SYMJEPI with one hand with your fingers behind the needle and slide the safety guard over the needle until it clicks. Put the used syringe into the grey case and close the case.

- Take the case with the used syringe with you when you go to see a healthcare provider.
- Tell the healthcare provider that you have received an injection of epinephrine. Show the healthcare provider where you received the injection.
- Give your used SYMJEPI syringe to your healthcare provider for inspection and proper disposal.

After Use
Safety Guard



Using one
hand with
fingers
behind the
needle slide
safety guard
over needle.

For more information see Patient Information
sheet or ask your healthcare provider.

How to Store - Keep SYMJEPI in its plastic case nearby and ready for use at all times.

- Store at room temperature between 68°F to 77°F (20°C to 25°C). Do not expose to extreme cold or heat. For example, **do not** store in your vehicle's glove box and **do not** store in the refrigerator or freezer.
- Store the SYMJEPI syringe in its closed plastic case to protect from light.

Check Your SYMJEPI Periodically -

The solution should be clear when viewed through the window on the device. If the solution is discolored (pinkish or brown color), cloudy or contains particles, replace the SYMJEPI with a new one.

Your SYMJEPI Has an Expiration Date -

Example 2021 12 = December 31, 2021. Replace SYMJEPI before the expiration date. Dispose of expired SYMJEPI properly by taking the SYMJEPI in its case to a healthcare provider or hospital emergency room.

For more information about SYMJEPI pre-filled syringe and proper use of the product, call 1-800-230-3935 or visit www.symjepi.com.

SYMJEPI is a registered trademark of Adamis Pharmaceuticals Corporation.

Manufactured for Adamis Pharmaceuticals Corp. San Diego, CA 92130
Distributed by USWM, LLC, Louisville, KY 40241.

This Instructions for Use has been approved by the U.S. Food and Drug Administration.
Revised: July 2020

SYMJEPI

(epinephrine
injection, USP)
pre-filled syringe

Instructions
for Use

**Pull out
and
Read
before
Use**
Rx Only

H

